

FLEXXIDISC
Operating Instructions



Horse Dental Equipment

www.horse-dental-equipment.com

Operating Instructions

Precautions

- Maintenance must be done at least once a year or every 500 horses
- Do not use this handpiece for something else than its intended purpose
- Disassembly of the flexidisc's head will void the warranty


The different models

| Model | Curved twist | Straight | Length (cm) |
|----------|----------------------------------|----------------------------------|-------------|
| Long | ✓ | ✓ | 54 |
| Medium | ✓ | ✓ | 50 |
| Standard | ✓ (with or without extension) | ✓ (with or without extension) | 32 |
| Short | | ✓ | 18 |

Different shapes & sizes adapted to all dental practices



Advantage n°1

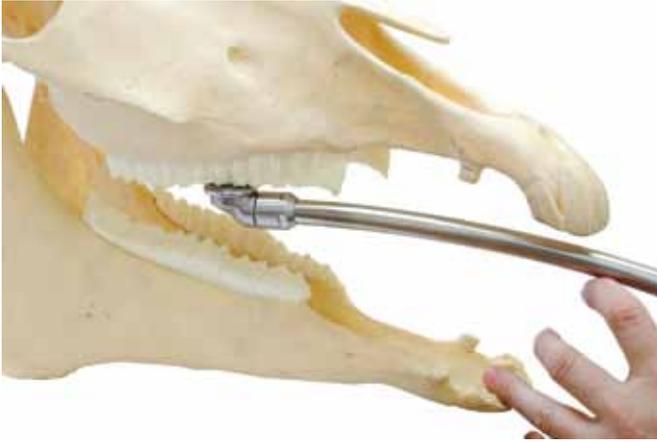

An efficient floating of the sharp enamel points thanks to its diamond disc.

Advantage n°2



The very slim head & body profile allows an easy access, far into the mouth.

The curved shape allows to see the disc working on the teeth, especially when working on the vestibular part.

Advantage n°3



Its rubber handle offers optimized confort and grip.


How to turn the TWIST head (curved model)

Step n°1



Hold both head + ring and pull **strongly** outward.

Step n°2

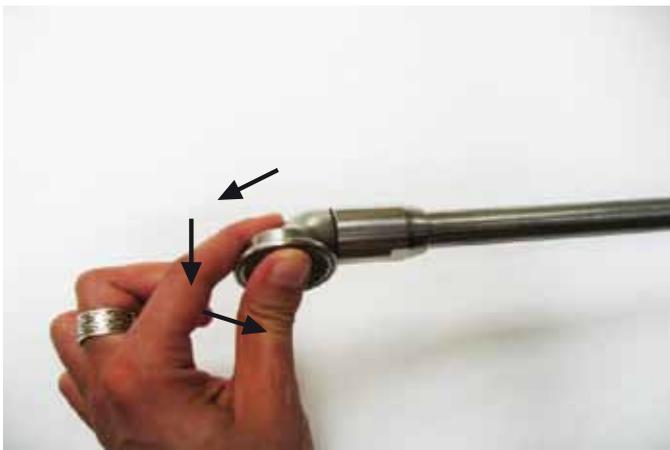

Turn gently and release.

Step n°3



The head is correctly positioned when it reaches the notch and you hear a click.

Cleaning recommendations

Diamond burrs

- 1 - Start the engine with the handpiece attached
 - 2 - Run it at high speed with the head pointing down to remove any remaining water
 - 3 - Clean the diamond burr by running it over the HDE stainless steel brush
 - 4 - Continue until the remains of dentine have completely disappeared
 - 5 - Clean with a cloth moistened with clean water and a disinfectant
 - 6 - Thoroughly dry with a clean, soft cloth
 - 7 - Store the burr in its dedicated place within your clean, dry case
- The burr is resistant to water projection, do not fully immerse in water**
Always use clean water without corrosive nor abrasive products
Only use the HDE stainless steel brush



Cleaning recommendations

Drive shafts

- 1 - Clean the outside sheath with a damp sponge
- 2 - Dry thoroughly with a clean, soft cloth
- 3 - When finished working, detach the drive shaft from the engine and the handpiece
- 4 - Store the drive shaft in your clean, dry case

Ensure not to bend the drive shaft in an excessive radius of curvature

Engine

- 1 - Turn off the engine before cleaning
- 2 - Clean the engine with a damp sponge
- 3 - Thoroughly dry the engine with a clean, soft cloth
- 4 - Store the engine in your clean, dry case

Case

- 1 - Clean the case with a damp sponge
- 2 - Thoroughly dry the case with a clean, soft cloth
- 3 - Store in a clean and well ventilated place

Make sure the case is opened from time to time, to ensure your equipment is well ventilated

For any problem, please contact your
local technical center

HORSE DENTAL EQUIPMENT

ZI de Bellevue - 14 rue Blaise Pascal
35220 CHATEAUBOURG
France
horse@horse-dental-equipment.com
+33-299-007-129