

Instructions FLEXXIDISC
Repair / Change
Inner Cable, Head & Bearing-Holder



Horse Dental Equipment

www.horse-dental-equipment.com

Repair FLEXXIDISC HDE

Tools & pieces to use (photo 1 & 2)

- 1 flat spanner 22
- 1 grip vise
- 1 clean towel
- 1 new inner cable for FLEXXIDISC
- 1 bearing holder
- 1 circular tube




TOOLS

Photo n°1

PIECES

Photo n°2

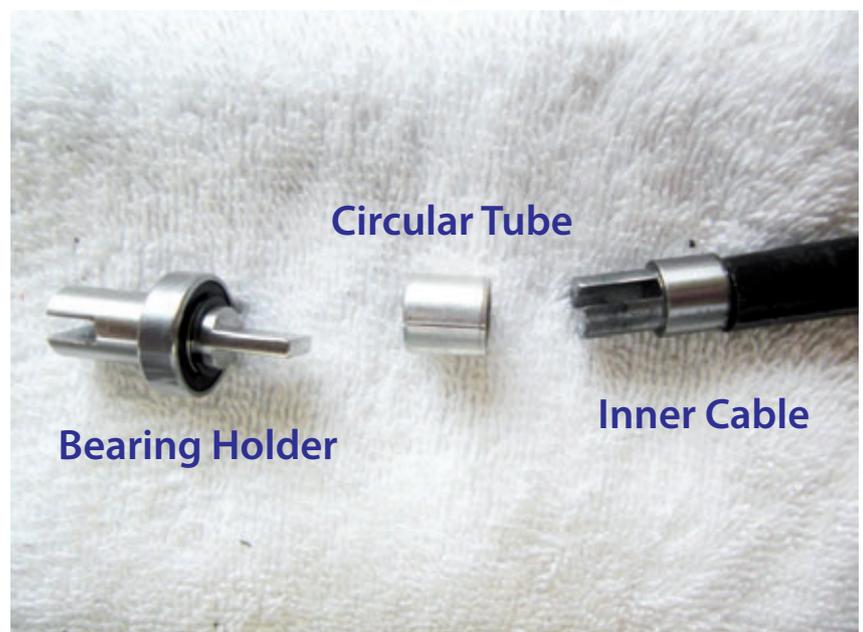

Step n°1

First, unscrew the cap of the body with the flat spanner no. 22 (photo 3), then, with the same spanner, unscrew the nut of the Flexxidisc head (photo 4).

Once those two pieces unscrewed, remove the bearing holder piece (pulling it out with a pincers) and then Flexxidisc head as the inner cable is screwed to it (photo 5).

Check the state of the bearing holder piece. If you think it is necessary to change it, please refer to stage no. 3.

Photo n°3



Photo n°4



Photo n°5

Step n°2

If you need to change the Flexxidisc head and/or the inner cable, you have to use the grip vise in order to jam the inner cable and unscrew the Flexxidisc head by holding the diamond disc with your hand (photo 6).

Replace the inner cable and/or the head.

Once the inner cable is perfectly screwed with the head, put it back into the body. Be careful to correctly place the teeth of the body in the Flexxidisc head (photo 7), and finally screw the nut.

Before replacing the bearing holder piece, it is indispensable to place the circular tube at the extremity of the inner cable (photo 2).



Photo n°6

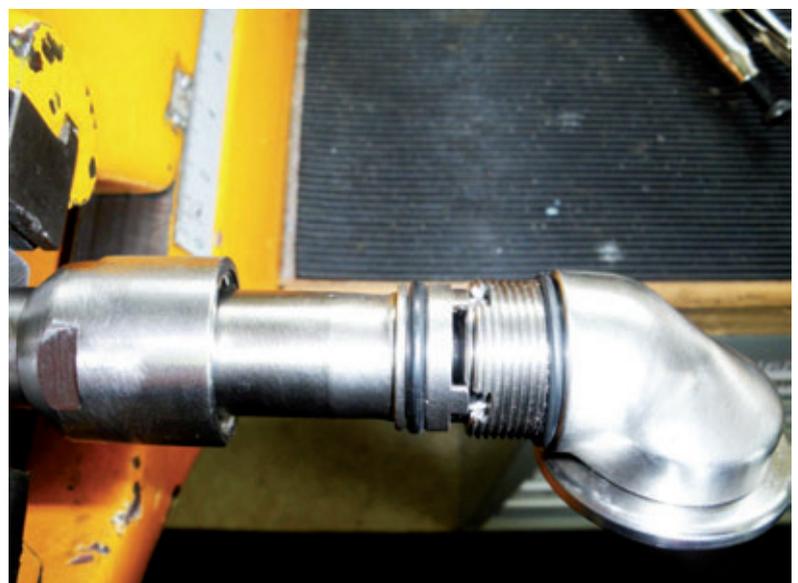


Photo n°7

Step n°3

If you need to change the bearing holder piece, you must clean its place in the body first and then put the new one (photo 8).

Screw the capp to the body and check everything works well by making the diamond disc turn with your hand.

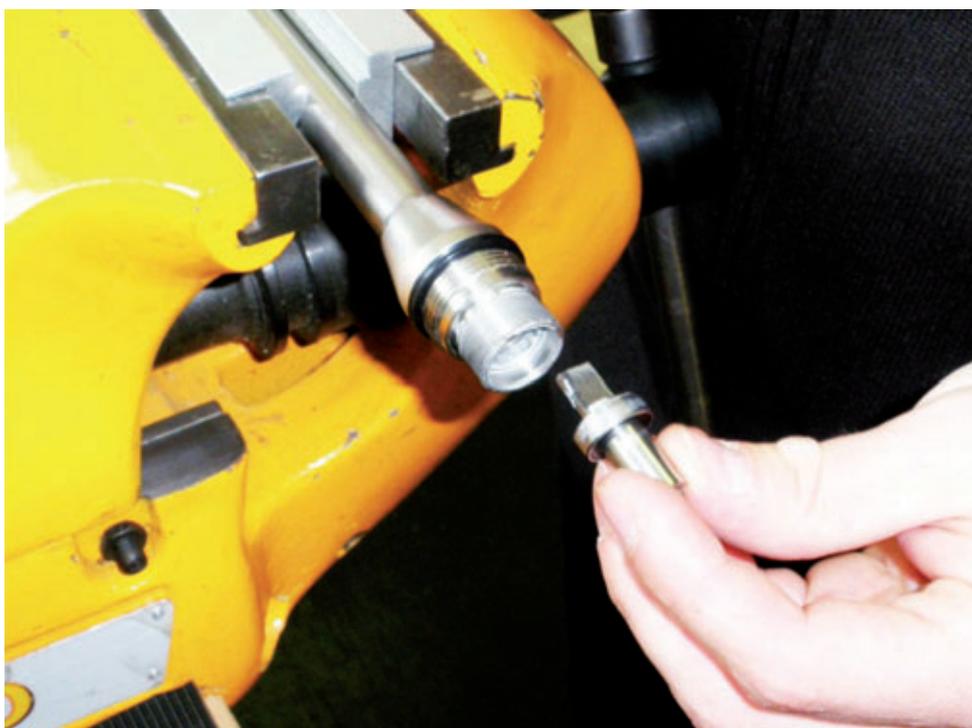


Photo n°8

Please visit our website for further details
including a video clip showing
the changing procedure:
www.horse-dental-equipment.com

Horse Dental Equipment

ZI De Bellevue, 8 Rue Blaise Pascal 35220 CHATEAUBOURG France

Tel : 00 33 (0) 2.99.00.71.29 - Fax : 00 33 (0) 2.99.62.30.91

E-mail : horse@horse-dental-equipment.com

www.horse-dental-equipment.com